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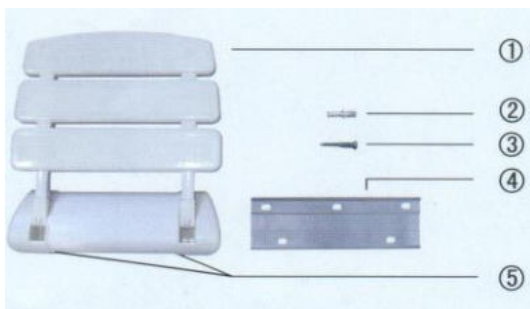
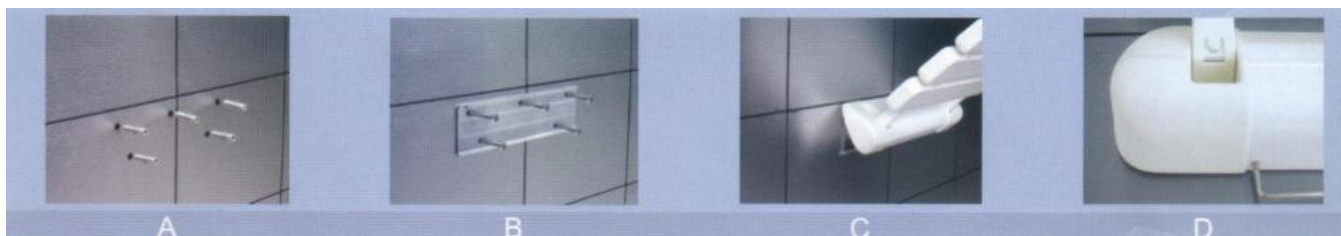
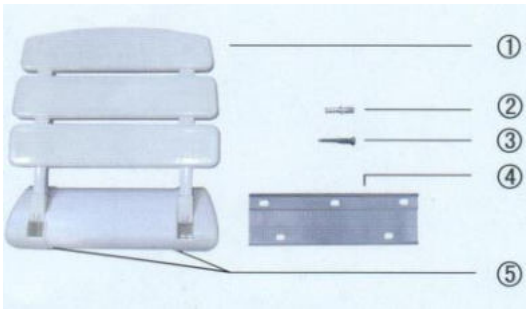
## Installation manual

### Installing steps:

**A:** Keep the plane of Fixing board (4) towards the wall and place it on the wall level, the edge with three holes of Fixing board (4) should be up. Recommended upright height from the floor is 450-550 mm. Mark down the screw holes and take down the Fixing board. Drill five holes with the depth of 60 mm and diameter of 8 mm and place the rawplug (2) into the holes.

**B:** Put the Fixing board (4) onto the original place once again and screw down the expand screw (3) and fasten them.

**C:** Button the Main body for shower seat (1) downwards absolutely from the upper part of Fixing board (4) until the inner side on the bottom of Main body for shower seat (1) touches the wall and keep the pedestal on the Main body for shower seat (1) and the two ends of Fixing board (4) orderly.

**D:** Screw down two pcs of Hex socket Grub Screw (5) on the underside of Main body for the shower seat (1) with the supplied Hex socket spanner.

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## GB

### **Important notes!**

The folding shower seat is designed for installation only in wet rooms. It absolutely must be attached to a stable wall that is capable of supporting a load and with a smooth surface. The fitter must ensure that the construction of the wall ensures the secure attachment of all wall mounting screws. Included standard mounting hardware: Nylon dowels, 8 x 60 mm and stainless steel screws, 5 x 57 mm, suitable for reinforced concrete or solid brick walls (M20). For other wall types, a specialist firm must use additional mounting techniques. Safety-tested up to 150 kg. The mounting height of the folding shower seat depends on the height of the user; the standard height is 450 - 550 mm, measured from the floor.

### **Cleaning and care instructions:**

1. To care for the product, use only commercially available cleaning products that are suitable for the product.
2. Do not use any abrasive, corrosive or solvent-based cleaners.

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